SKIVEL – User manual

TRIVEL

The Skivel is designed to let you enjoy winter with friends, or with a wheelchair-bound patient. The secure attachment system and sturdy handle ensures you are always in full control of your movements. Made to be used on groomed snow.

WARNING

- Slow down before turning to avoid tipping over.
- Use exclusively on flat, snow-covered surfaces.
- Do not use on steep inclines as the Skivel and wheelchair could tip.
- Do not use on uneven sidewalks or near stairs.
- Each quick release that holds the Skivel frame in place once adjusted must be securely tightened.

1. Place the Skivel on a flat surface. Loosen the quick release handle at the base of the handlebar. Position the handle in the best possible position for the user to push and retighten the quick release to hold it in place.

2. Measure the inside tires width of the wheelchair that will be used.

3. Use the eight quick releases handles to adjust the width of the Skivel to the inside width of the wheelchair wheels.

4. Adjust the distance between the rollers with an Allen key if necessary. They must hold the wheelchair in place and prevent it from moving.

Before use:

- Make sure wheelchair wheels are locked before moving.
- The wheels of the wheelchair must be in the center of the skis.
- The wheelchair must face away from the handlebar.
- Maximum total weight capacity is 400lbs/182Kg.
- Do not leave a user on the Skivel unattended.

Feel free to contact us at 1-877-687-7676, if you need assistance with your Skivel.

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